FITTING PROCEDURE

ENGLISH | ESPAÑOL | DEUTSCH | ITALIANO

www.walkease.co.nz

Distributed by Shoof International Ltd, PO BOX 522
Cambridge 3450, New Zealand

FITTING PROCEDURE

For use as early-intervention hoof treatment

Treat the injured claw as required. Treatment of the injured claw must be completed first and then the healthy claw is prepared for fitting the block (photo 1).

Use the Walkease® hoof rasp to prepare the healthy claw for gluing. The preparation involves using the Walkease® rasp supplied in the Walkease® kit. We recommend using this rasp to finish the healthy claw surface even if a grinder or other cutting tool has been used to prepare the claw. The Walkease® rasp provides the correct flat, clean and dry surface for the Walkease® block to adhere to (photo 2). The rasp must also be clean and dry before use. Do not contaminate the hoof surface in any way before applying a Walkease® block.

Test the Walkease® block for size and shape on the healthy claw. Choose the correct size that will support the whole hoof wall. Handle the Walkease® block carefully ensuring the gluing face is kept perfectly clean and dry. Every Walkease® block is individually wrapped in plastic to ensure that it is kept clean before use. If accidentally contaminated it can be washed with hot water and detergent, or with alcohol, then dried completely before using. To avoid getting glue on hands, it is essential that you wear gloves during this procedure. Apply glue sparingly to the Walkease® block (photo 3). Do not use excessive glue (2ml only).

Press the block firmly onto the healthy claw. The block should overlap the hoof wall by 2 or 3mm. This will give support under the hoof wall, where most load bearing is required (photo 4). After about 10 seconds of steady pressure you will feel the block 'grab' (Possibly longer in cold weather).

Drop the leg for the cow to put weight on the block. The Walkease block will not attach properly if weight is not applied before the glue has completely set. **(This part is most important).** The glue sets very quickly under the animal's weight (photo 5). The excess glue is squeezed out and the block conforms to the claw surface. This improves the surface area contact giving a stronger bond. We recommend that any surplus block outside the outer edge of the hoof is trimmed off after the hoof is put down. This will reduce the risk of the excess block being caught under another cow's hoof or other obstruction. In colder weather the bond time may increase a little, so give the animal a full minute standing on the block before releasing it from the crush.

Extra Blocks and glue can be ordered through your Shoof dealer.


Walkease® Kit

The kit contains ten Walkease® blocks, enough special Walkease® glue to apply, a Walkease® hoof rasp and ten pairs of gloves for use during application.

Walkease® Refill

The Walkease® Refill package contains ten Walkease® blocks, to be used with Walkease® glue, for blocking cow hooves.

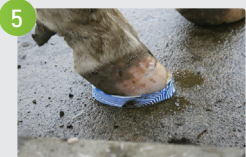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EU: distributed by Albert Kerbl GmbH, Felizenzell 9, 84428 Buchbach